


# HOW TO WEAR YOUR CUSTOM PRODUCT

## STEP 1

The small **Red** marker indicates the right side. There may be a **Blue** marker for the left. Apply a light coating of the ear lubricant directly to the earmold.

**DO NOT** plug the canal vents with lubricant.

## STEP 4

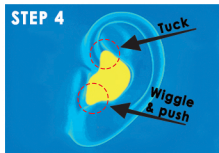


Tuck top of earmold under helix pocket, and gently wiggle canal to seal.

## STEP 2




Place the canal of earmold into ear with the top of the earmold facing forward.

## STEP 5

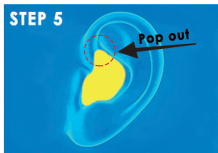


To Remove: Gently pop out top portion of earmold from pocket of ear.

## STEP 3

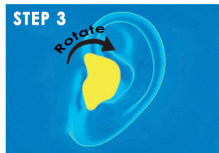

Rotate earmold back while simultaneously pushing it into the ear.

## STEP 6

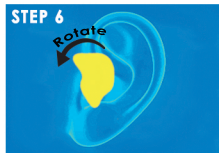

Rotate the earmold forward towards the nose. **DO NOT** Pull on Wires or cords!